

Select tab for patients info:

- Manual entry
- Worklist

Tabs of:


- digitization
- archive of images and videos

Tabs of:

- videosource
- Edit videos and pictures

Buttons for:

- Capture images
- Start/Stop video
- Enable/Disable audio

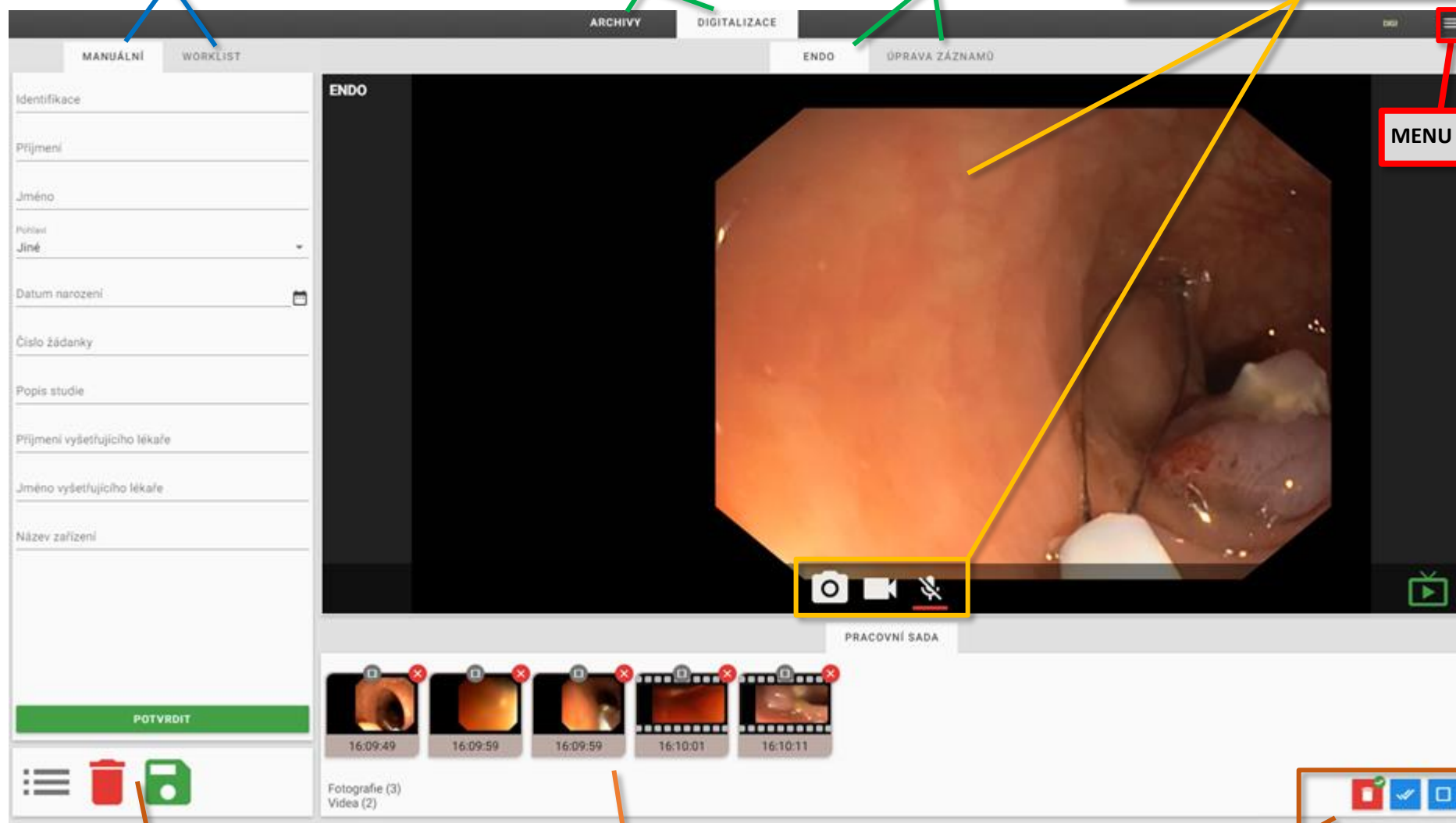

Buttons for:

- Queue of sending records
- DELETE actual records and patients info
- SAVE actual records with patient info to PACS

WORKSET:

- Actual records (images and videos)

Buttons for:

- DELETE mark records in workset
- MARK/UNMARK records in workset

Step 1 – Start device - At the front of device push the button and wait 1 minute - The PC and the software Dicompass Gateway start automatically - Wait for login screen and use your login and password to system	Step 2 – Patient info - Use manual entry and fill all the patient information or use tab Worklist and select patient - Required is ID, Surname and name - After filling patient information click to button „CONFIRM“  - For edit patient data click to button „Edit“ 	Step 3 – capture images and videos - For capture images and record videos use this button on the screen  - Or use the USB foot switch - Green pedal is for images - Orange pedal is for record video (Start/stop) - Workset show you in second your capture images and recording videos
Step 4 - Uložení - After you finish the examination it is necessary SAVE or DELETE the records: - For saving use this button  - For deleting use this button 	Step 5 - Shutdown - Select tab „Control“ next to videosource and edit records  Red Button – Shut Down PC Yellow Button – Shut down PC after sending all records to PACS – best choice Blue button – restart PC	